Manufactured by Hyphen BioMed.

# BIOPHEN HEPARIN ON SYSMEX CA 7000

## Adaptation of BIOPHEN HEPARIN ON SYSMEX CA 7000

### 1. Reconstitution of the BIOPHEN Heparin 6 reagent, (Reference A221006)

Chromogenic determination of the Anti-Xa activity of LMWH, UFH, and Orgaran.

	NAME	Reconstitution	Stability	Stabilization in T°
R1	SXa-11 Substrate	7.5 ml of distilled water (*)	3 months at 2-8°C ** 7 days at room T° Do not freeze	30 mn <i>on board</i> before any use(***)
R2	Factor Xa	7.5 ml of distilled water (*)	3 months at 2-8°C ** 7 days at room T° Do not freeze	30 mn <i>on board</i> before any use (***)
Diluent	Physiological Saline		24 hours <i>on board</i>	

### 2. Reconstitution of the BIOPHEN Heparin 3 reagent, (Reference A221003)

Chromogenic determination of the Anti-Xa activity of LMWH, UFH, and Orgaran.

	NAME	Reconstitution	Stability	Stabilization in T°
R1	SXa-11 Substrate	3.75 ml of distilled water (*)	3 months at 2-8°C ** 7 days at room T° Do not freeze	30 mn <i>on board</i> before any use(***)
R2	Factor Xa	3.75 ml of distilled water (*)	3 months at 2-8°C ** 7 days at room T° Do not freeze	30 mn <i>on board</i> before any use (***)
Diluent	Physiological Saline		24 hours <i>on board</i>	

Recommendations for reconstitution:

**Reconstitution (\*):** Reconstitute each vial with distilled water. Shake thoroughly (vortex). Let to homogenize for 30 minutes at room temperature (18-25 °C), while shaking the vial from time to time (vortex), until complete dissolution of the content. Check the absence of any solid at the bottom of the vial.

In current practice, in order to allow a good standardization, reconstitute these two reagents the evening before and put them at 2°-8°C following the 30 minutes at room temperature.

**Note: In all cases, before use, check the absence of solids at the bottom of the vial, which confirms that dissolution is complete. If necessary, incubate for 1 hour at RT or better at 37°C, while shaking (vortex) from time to time. If required, then additionally incubate overnight at RT.**

**Storage of reagents: (\*\*)** Stability indicated are validated provided any evaporation or contamination of reconstituted reagents avoided. They must be adjusted according to the actual laboratory working conditions

Take care of putting up the specific caps back on the bottles before storing them at 2°-8° C, and of strictly respecting the temperature stabilization time of 30 minutes before using the reagents on the automate.

If the reagents are kept on the automate board 24H/24H, take care to limit as much as possible any evaporation of the reagents by using a chimney or mini-reducer.

**Stabilization of reagents: (\*\*\*)** It is necessary to let the substrate and the Factor Xa temperature to stabilize for at least 30 minutes on the automate before any use. A too low temperature of the reagents can induce an over-estimation. Conversely a too high temperature leads to an under evaluation of heparin.

**Homogenize the reagents before each use.**

Any reagent of biological origin must be handled with all the required cautions, as being potentially infectious.

**Do not interchange the reagents from different lots.**

**If necessary use microcups when running the assay, to improve management of the reagents volumes by the automate**

### 3. Determination of Heparins UFH and/or LMWH.

NAME	Reconstitution	Stability	Stabilization in T°
Calibration Biophen Heparin Calibrator (réf A222001)	1 ml of distilled water (*)	7 days at 2-8°C 48 hours at room T° Do not freeze	30 minutes <i>on board</i> before any use (**)
Quality controls Biophen UFH Control (réf A223101) Biophen LMWH Control (réf A223001)	1 ml of distilled water (*)	7 days at 2-8°C 48 hours at room T° Do not freeze	30 minutes <i>on board</i> before any use (**)

### 4. Determination of Orgaran:

The determination of Orgaran requires its own configuration of the CA 7000. This one differs from the configuration used for the *determination* of UFH/LMWH only by the denomination of calibrators and controls.

NAME	Reconstitution	Stability	Stabilization in T°
Calibration Biophen Orgaran® Calibrator (réf A222201)	1 ml of distilled water (*)	7 days at room T° 48 hours at room T°	30 minutes <i>on board</i> before any use (**)
Quality controls Biophen Orgaran® Control (réf A223501)	1 ml of distilled water (*)	7 days at room T° 48 hours at room T°	30 minutes <i>on board</i> before any use (**)

#### Reconstitution:

(\*) After reconstitution of calibrators or controls with distilled water, let them to stabilize for 30 minutes at room temperature. It is better to reconstitute calibrators the very day of calibration.

### Conservation of reagents:

(\*\*) Take care of strictly respecting the 30 minutes temperature stabilization time for *calibrators* and *controls* at room temperature, then the 30 minutes on the automate, particularly if they were stored at + 2°-8°C. Homogenize before each use.

**Nota:** For lyophilized calibrators and controls, following reconstitution with distilled water, let the reagent to stabilize 30 minutes at room temperature. It is recommended to run the calibration curve with a freshly reconstituted calibrator. It is necessary to let the reagent temperature to stabilize for at least 30 minutes onto the automate before any use. Take care avoiding any contamination or evaporation of the reagents. Stability can be adjusted according to the exact use conditions.

**Homogeneize before each use.**

**Do not freeze calibrators and quality control plasmas.**

**A new calibration curve must be carried out for each new batch of reagents, after each important maintenance of the instrument, or when measured values for controls are out of the acceptance range for the method (after checking all other parameters for the system).**

**Performances may present slight variations according to the instrument used. Validate the expected values in the laboratory working conditions.**

**Check the validity of the series by including quality control plasmas at different levels in each one.**

**Quality controls must be run regularly, and for each new batch of reagents, after an important maintenance of the instrument, or if measured values are not in compliance with the one expected for the method.**

### **5. Results:**

- The calibration curve is of the Log (absorbance) - Lin (concentration) type.
- The values obtained for the patients and controls are directly calculated from the calibration curve.
- The results are expressed in IU/ml.
- When Heparin or Orgaran concentrations are out of the working range, assayed plasma must be diluted in normal plasma, appropriately prepared and platelet poor, in order to keep a sufficient concentration of AT III.
- In presence of low AT III concentrations, as it can be the case in young children, an exogenous source of AT III is necessary, in order to correctly measure the heparin concentration.

**Nota:** Biophen Heparin reagents are developed for measuring homogeneously Unfractionated heparin (UFH) and Low Molecular Weight Heparin (LMWH), using the same calibration curve.

The superimposition conditions are susceptible to slightly vary according to the combination of reagents and lots used, and the technical characteristics and specific adjustment of each apparatus.

Therefore, this superimposition has to be verified and validated in the exact laboratory working conditions and for each machine, by establishing the calibration curve using Biophen Heparin calibrators (LMWH, ref A222001), and by assaying the homogeneous system of associated Biophen UFH calibrators (ref A222301), UFH control plasmas (ref A223101), and LMWH control plasmas at high and low levels (ref A223001, A223701).

The linearity (r<sup>2</sup>) is expected to be ≥0.98 for the calibration curve.

Controls must be measured in compliance.

Measured values for calibrators are acceptable when:

Measured value = target value  $\pm 0.10$  IU/ml for levels  $\leq 0.50$  IU/ml (better  $\pm 0.05$  IU/ml)  
Measured value = target value  $\pm 0.15$  IU/ml for levels  $> 0.5$  IU/ml (better 90-110% of the target value, or  $\pm 0.10$  IU/ml for levels between 0.5 and 1.0 IU/ml)

Should the superimposition not be obtained, a separate calibration curve must be used for each type of heparin, and validated by assaying the homogeneous quality control system (controls measured in compliance and close to the assigned target value).

The calibration curve is validated when linearity ( $r^2$ ), as well as measured control values, are in compliance.

A new calibration curve must be carried out for each new batch of reagents, after each important maintenance of the instrument, or when measured values for controls are out of the acceptance range for the method (after checking all other parameters for the system).

Performances may present slight variations according to the instrument used. Validate the expected values in the laboratory working conditions. Performances, as well as values for each new lot of quality controls used, must then be confirmed (and adjusted if necessary) in the laboratory working conditions.

**Nota:** The 3 months stability at 2-8°C and the 7 days stability at room temperature (18-25°C) claimed are validated on reconstituted vials kept closed, protected from any evaporation or contamination.

For your information, stability data on STAR board tested in our laboratory working conditions are:

-7 days for reconstituted vials loaded on STAR during working time, using mini reducers or chimneys then closed and stored at 2-8 °C overnight.

-5 days for reconstituted vials left loaded continuously on STAR.

Stabilities of reagents on each automate must be controlled, adjusted and validated under each exact laboratory working conditions.

## Instrument setting for Sysmax CA 7000

### Create the reagents

Go to Analysis Configuration

Then Reagent information

Create the reagents

*Example of creation for Biophen Heparin 6 reference 221006:*

ID	NAME	LOT	Code Bar	Expir	Limit Init Vol.	Limit Alert	Intern Limit	Vial
1006	Sub Hep	40702- 1		11/07	7.5 ml	1.02		GW5
1007	FXa Hep	40702- 2		11/07	7.5 ml	1.02		GW5
1008	Eau Phy	0		0	5 ml	1.02		GW5
1009	Cal 1....	40703		11/07	1 ml	0.2		Cup2

**ID** can be created with 4 numbers.

**Name:** Sub Hep or FXa Hep or eau Phy (NaCl must be in vial of 5 ml as the other reagents).

**Limit Alert:** 1.02 ml for the vial containing 5 ml and more.

**Vial** = GW5 for the vial of 5 ml or more and Cup2 for Standards and controls (vial of 1 ml).

**Go to « Protocole test » and write the following informations:**

Click on the window Set up software for the “manager program” and create the program according to: Chromogen for HEPARIN

Change the data with the data indicated on the table below

Use the program and create the name for the reagents, Calibration plasmas and control plasmas

**PROTOCOLE FOR 333 TESTS:**

System		Ready			
				Emergency	
<b>Analysis protocol</b>					
<b>Parameter : UFHLMWH Para Code 44</b>					
Sample Vol.			<b>18**</b>	$\mu$ l	
Vol. diluent	<b>NaCl</b>		<b>36**</b>	$\mu$ l	Detection method
Washing	Sans				<b>1</b>
Second dilution	No	0 $\mu$ l			7
Diluent Vol.	Without	0 $\mu$ l			8
Washing		Without			9
<b>Deficient Plasma</b>	Without	0 $\mu$ l	0 sec	Norm	
Vol Diluent	No Util.	No	0 $\mu$ l		4
Washing	Without	x0/Off/Without x0			5
<b>First Reagent</b>	<b>SUBhep</b>	<b>90<math>\mu</math>l**</b>	<b>12 sec.</b>	Norm	
Diluent Vol	Needle R A2	No	0 $\mu$ l		1
Washing	Without	x0/	Clean I	x 1	2
<b>Second Reagent</b>	<b>XaHep</b>	<b>90<math>\mu</math>l**</b>	<b>252 sec.</b>	Norm	
Diluent Vol	Needle B1	No	0 $\mu$ l		0
Washing	Without	x0/	Clean I	<b>x3</b>	Enter
<b>Third Reagent</b>	Without	0 $\mu$ l	0 sec	Norm	
Diluent Vol	No util	No	0 $\mu$ l		
Washing	Without	x0/Off/Without x0			
Detector (*)	Chromogen For AT III				
Wavelength (*)	Bas sens /405nm		Inc		
Detection setting (*)	20 sec		60 sec		
Select Tests	Test Name	Special	↑	↓	Return

(\*) For CA7000 ⇒ These informations are mentioned at the beginning of the table

**PROTOCOLE FOR 375 TESTS: (\*\*) 16 $\mu$ l sample Vol, 32  $\mu$ l Diluent Vol, 80 $\mu$ l Sub, 80 $\mu$ l Xa**

**Go to « Analysis Configuration »:**

Click on “Calcul Parameter” and complete the following table:

System		Ready analysis	HC GP		
			Emergency		

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