Manufactured By: HYPHEN BioMed

HEMOCLOT T.T.

on STA

Determination of THROMBIN TIME with a clotting method

Adaptation of HEMOCLOT T.T. ON STA

1. Reconstitution of HEMOCLOT T.T. (Ref ACK011K 6x2ml or ACK011L 6x8ml) reagent.

Unopened reagent must be stored at 2-8° C, in their original packaging box. They are stable until the expiration date printed on the label.

	NAME	Reconstitution	Stability	Stabilization at room Temperature
R1	Bovine Calcium Thrombin	2 ml (ACL011K) or 8ml (ACK011L) of distilled water (*)	48 hours at room T° 7 days at 2-8° C	30 min on STA board before any use (**)
Diluent	Owren Koller		24 hours on STA board	

Reconstitution: (*) Following reconstitution with distilled water, let the reagent to stabilize 15 minutes at room temperature.

Stabilization of reagents: (**) It is necessary to let the reagent temperature to stabilize for at least 30 minutes on the automate before any use.

NOTE: Plasmas used for the Thrombin preparation were tested with registered methods and found negative for HIV antibodies, HBs Ag and HVC antibodies, or BSE. However, no assay may warrant the total absence of infectious agents. Any product of human origin must then be handled with all the required cautions as being potentially infectious.

2. Reconstitution of controls:

The control is performed with commercially available control plasmas.

Nota : Following reconstitution with distilled water, let the reagent to stabilize 15 minutes at room temperature. It is necessary to let the reagent temperature to stabilize for at least 30 minutes onto the automate board before any use.

Do not freeze quality control plasmas

3. Results:

- The values obtained for the patients and controls are directly expressed in seconds.

Programming the STA analyser

Click on the icon **set up software** for the **manager program** and create the program HEMOCLOT T.T. according to:

CONFIGURATION – Page 1 / 3

Identification									
Abbreviation		Name			Method			Date	
T.T		HEMOCLOT T.T			CLOT-BASED				
Sample				Diluent					
Volume	Incu	Dil.	Id.	Name		vial.	stab	Min.Vol.	
100µl	240 sec	1/1	OK	OWREN KOLLER		15 ml	72h	0.2 ml	
Reagents									
Id.	Name	Incu sec	Vol. µl	Vial ml	Stab h	Min. Vol. ml	Washing		
							Before	after	
Ra									
Rb									
Rc									
Rd T.T	THROMBIN		100	2 or 8	48	0.2	no	Special	
Analysis		Result				Validation (%)			
Min Time	10 sec	Primary Units		Sec		Standard	min	max	
Max Time	120 sec	Corrector		1.00			10	120	
Mean Time	20 sec	Single/Duplicate		single					
Rd Heating	yes	Precision		5%					
Stirring	yes	Redil. Condition							
Clot Type	normal	<							
		>							

Enter data concerning washing.

Washing				
	Ra	Rb	Rc	Rd
Before				No
After				Special
Wash				12227
Name				STA-DESORB.U
Stab. H				120
Vial ml				15
Vol in. ml				0.9

Show the second page and write calibration data.
No calibration for Hemoclot T.T.

Calibration						
Mode RAW						
Results in SEC						
Control						
	Id.	Key	Name	Vial. ml	Stab. H	Min Vol ml
Level 1						
Level 2						

Go to the last page and enter data concerning the quality control.

The STA can manage 3 levels of quality control. For exemple: the following configuration is possible.

Printout / Transmission							
Units	Converts Factors.	Print	Transmission Test Number	Usual Values	Sec		
Main	sec	yes	0	STANDARD	Min.	Max.	
Aux.1			0		15(*)	25(*)	
Aux.2							
Aux.3							
Printout limits		min	10				
		max	120				
Controls							
	Id.	Key	Name	Period H	Vial. ml	Stab. H	Min Vol ml
Level 1	(*)	(*)	(*)	(*)	(*)	(*)	(*)
Level 2	(*)	(*)	(*)	(*)	(*)	(*)	(*)
Level 3	(*)	(*)	(*)	(*)	(*)	(*)	(*)

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