**Adaptation of BIOPHEN HEMOCLOT VII-X ON BCS**

**1. Reconstitution of BIOPHEN HEMOCLOT VII-X (Ref ACK051K) reagent.**

Determination of factors VII-X activity with a clotting method

NAME	Reconstitution	Stability	Stabilization at room Temperature
Hemoclot VII-X Reagent	1 ml Distilled water (*)	24 hours at room T° 72 hours at 2-8°C 1 month frozen at -20°C	30 mn before any use (**)

**Reconstitution:**

(\*) After reconstitution with distilled water, let the factor VII-X reagent to stabilize 15 minutes at room temperature.

**Conservation of reagents:**

Take care of putting up the specific caps back on the bottles before storing them at 2°-8° C and of strictly respecting the temperature stabilization time of 30 minutes before using the reagents on the automate.

**Stabilization of reagents:**

(\*\*) It is necessary to let the reagent temperature to stabilize for at least 30 minutes on the automate before any use.

**Preparation of the calibrators and controls**

**Calibration curve:**

- Prepare 1 ml of normal human pooled plasma diluted 1:10 in Owren Koller buffer (by definition the normal citrated plasma pool corresponds to a concentration of 100% of Factors VII + X).

Or

- The calibration curve can also be established with the BIOPHEN Plasma Calibrator (# A222101), using the factor VII-X activity indicated for the lot used.

Name	Reconstitution	Stability	Stabilization in T°
CALIBRATION Biophen Plasma Calibrator (ref A222101)	1 ml of distilled water(*)	24 hours at 2° - 8°C 8 hours at Room T°	30 minutes <i>on automate board</i> before any use (**)

## Controls

Nom	Reconstitution	Stability	Stabilization in T°
CONTROLES DE QUALITE <u>Biophen Normal Control</u> (ref A223201) <u>Biophen Abnormal Control</u> (ref A223301)	1 ml of distilled water(*)	24 hours at 2° - 8°C 8 hours at Room T°	30 minutes <i>on automate board</i> before any use (**)

### Reconstitution:

(\*) After reconstitution with distilled water, let the reagent to stabilize 30 minutes at room temperature.

### Stabilization of reagents:

(\*\*) It is necessary to let the reagent temperature to stabilize for at least 30 minutes on the automate before any use.

### Conservation of reagents:

Take care of putting up the specific caps back on the bottles before storing them at 2°-8° C and of strictly respecting the temperature stabilization time of 30 minutes before using the reagents on the automate.

Do not freeze calibrators and controls

**Nota : A calibration curve must be carried out of each new batch of Hemoclot Factor VII-X reagents**

**NOTA: Plasmas used for the pool and Thrombin preparation were tested with registered methods and found negative for HIV antibodies, HBs Ag and HVC antibodies. However, no assay may warrant the total absence of infectious agents. Any product of human origin must then be handled with all the required cautions as being potentially infectious.**

### Results:

The calibration curve is of the Log (% activity VII-X) - Log (Clot time) type.

The values obtained for the patients and controls are directly calculated from the calibration curve.

The results are expressed in % activity, the 100% activity being the activity in reference normal pool plasma.

## 5. Programming of the BCS analyser

Click on the icon **set up software** for the **manager program** and create the program according to:

### Creation of reagents files

Return to definition of test. Go into reagents without data of lot. Create the reagent files.

Name:	R1HEMOCLLOT VII-X		
Abbreviation:	R1 VII-X		
clean when the reagent is modified		Normal	
<input checked="" type="checkbox"/>	New calibration with any new lot of reagents		
<b>Authorized lines</b>	from <input type="text" value="1"/> to <input type="text" value="14"/>		
<b>Stability (*)</b>	< 15C <input type="text"/> > = 15C <input type="text"/>		
<b>Mix by jet</b>	Intensity <input type="text" value="1"/> Frequency <input type="text" value="No agitation"/>		
<b>Reference numbers</b>	Number <input type="text"/>		
	add	remove	replace

### Creation of controls files

Name:	HEMOCLLOT VII-X		
Abbreviation:	BNC		
clean when the reagent is modified		Low	
<input checked="" type="checkbox"/>	New calibration with any new lot of reagents		
<b>Authorized lines</b>	from <input type="text" value="1"/> to <input type="text" value="14"/>		
<b>Stability (*)</b>	< 15C <input type="text"/> > = 15C <input type="text"/>		
<b>Mix by jet</b>	Intensity <input type="text" value="1"/> Frequency <input type="text" value="No agitation"/>		
<b>Reference numbers</b>	Number <input type="text"/>		
	add	remove	replace

Name:	HEMOCLLOT VII-X		
Abbreviation:	BANC		
clean when the reagent is modified		Low	
<input checked="" type="checkbox"/>	New calibration with any new lot of reagents		
<b>Authorized lines</b>	from <input type="text" value="1"/> to <input type="text" value="14"/>		
<b>Stability (*)</b>	< 15C <input type="text"/> > = 15C <input type="text"/>		
<b>Mix by jet</b>	Intensity <input type="text" value="1"/> Frequency <input type="text" value="No agitation"/>		
<b>Reference numbers</b>	Number <input type="text"/>		
	add	remove	replace

(\*) To be filled by the user

Creation of the procedure

General Information			
Former Change			
Test Procedure Number			
Test Procedure Name	HEMOCLLOT VII-X		
Terminate measurement after	250 sec	Or	mA (0 = any measurement time)
	0		
Speed	normal		
Mixing	normal		
Wave-Lenght	405 nm		
Primary absorbance range	201-240 mA		
Duplicate # for samples and controls	1	CV Authorized	0.0 %
Duplicate # for calibration	1	CV Authorized	0.0 %
Duplicate # of raw data summary	Arithmetic mean		
Dilution Factor	1 : 4	(1 = no dilution)	
Dilution Buffer	OWREN KOLLER		

Evaluation and Control Methods			
<b>Fixed Absorption</b>			
Lag Phase	7	Sec	Threshold 100 mA
<b>Angle</b>			
Lag	7	Sec	Threshold 100 mA
Window size	3	Sec	
Minimun angle	30.0	angle	
<b>Decrease</b>			
Lag	7	Sec	
Max Absorbance Decrease	10	%	
<b>Sinusoide</b>			
Lag Phase	7	Sec	Threshold 100 mA
Minimum Amplitude	100	mA	Threshold Factor 2.0
Number of Steps	10		Limit 20.0

## Pipetting Sequence

Test Procedure Number

Test Procedure Name

HEMOCLOT VII-X

Sampling cycles

Number of Cycles	Transmission Arm	Final Washing	Rotors/transfert Action	Time Window [sec]	
				Min	Max
1	Arm	normal	No mix	0	0
2	Reagent Arm	normal	No mix./Measure	0	0

Transm. Number	Mean/pipetting purpose	Reagents	Speed	Volume (µl)	
1	1	Take air	Slow	20	
	1	Take the reagent	R1 VII-X	Slow	30
	1	Take the sample		Slow	20
	1	In the external cup		Slow	50
2	2	Take air	Slow	20	
	2	Take the Reagent	R2 THRO	Slow	150
	2	In the central cup		Slow	150

Test Definition					
<b>General Information</b>					
Former Change <input style="width: 150px;" type="text"/>					
Name		<input style="width: 150px;" type="text" value="HEMOCLOT VII-X"/>		°Test N° <input style="width: 50px;" type="text"/>	
Result Unit		PC ID t		<input style="width: 50px;" type="text"/>	
Behring	%	Conversion factor		Abbreviation	<input style="width: 50px;" type="text" value="VII-X"/>
User	%	1 %dN	<input style="width: 30px;" type="text" value="1.0"/>	%dN	Place in the liste <input style="width: 50px;" type="text"/>
PC	%	1 %dN	<input style="width: 30px;" type="text" value="1.0"/>	%dN	
Digit number for results					<input style="width: 50px;" type="text" value="100.0"/>

### Test detailed procedures

Test Procedure Name	Evaluation Method
HEMOCLOT VII-X	Delta A fix

### Calculate Raw Data Formula

Formula Type	Minimum <input style="width: 50px;" type="text"/>	<input style="width: 100px;" type="text"/>
User own's Formula	<input style="width: 100%; height: 20px;" type="text"/>	
Raw data unit	Sec <input style="width: 50px;" type="text"/>	<input style="width: 100px;" type="text"/>

### Detailed test evaluation

<b>Reference range</b>	Result Determination	<b>Standard Concentrations</b>
	<input style="width: 40px;" type="text" value="10"/> % dN at <input style="width: 40px;" type="text" value="150"/> % dN	<input style="width: 50px;" type="text" value="100"/>
<b>Test used for Calibration</b>	<input style="width: 150px;" type="text"/>	<input style="width: 50px;" type="text" value="50"/>
<b>Evaluation</b>	<input style="width: 150px;" type="text" value="Regression Log-Log"/>	<input style="width: 50px;" type="text" value="25"/>
<b>Reference Curve</b>	<input style="width: 150px;" type="text" value="Calibration curve"/>	<input style="width: 50px;" type="text" value="12.5"/>
		<input style="width: 50px;" type="text" value="6.25"/>
<b>Minimum raw value</b>	<input style="width: 60px;" type="text" value="0.0"/> Sec	<input style="width: 50px;" type="text"/>
<b>Upper Extrapolation</b>	<input style="width: 60px;" type="text" value="2.0"/> x maxi concentration	<input style="width: 50px;" type="text"/>
<b>Lower Extrapolation</b>	<input style="width: 60px;" type="text" value="0.5"/> x mini concentration	<input style="width: 50px;" type="text"/>
<b>Calibrator</b>	Calibrator <input style="width: 100px;" type="text"/>	<input style="width: 50px;" type="text"/>
<b>Authorized SD respectively to the reference curve</b>	<input style="width: 40px;" type="text" value="0"/> %	<input style="width: 50px;" type="text"/>
<b>Maximal number repeats</b>	<input style="width: 40px;" type="text" value="0"/>	<input style="width: 50px;" type="text"/>

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