**Determination of FVII activity by clotting assay**

**Indicative method Hemoclot FVII reagent on ACL7000 (research software)**

**NOTE :** reconstitution, preparation, calibration and controls, expression of results: usual cautions and realization according to the specific technical insert D.750.02/ACK/081K.

**Protocol:**

PHASE	CONDITION	OPERATION	PARAMETERS
1 <sup>st</sup> loading step	YES	Reagent loading (pos 18)	0µl
		Washing after reagent loading (pos 18)	0
		Diluent (pos DIL)	0µL
		Specimen (already diluted 1:10)	50µl
		Reagent (nacelle 2) (Hemoclot FVII)	50µL
		Washing between specimen	2
		Optical ref. (cycle abs.)	
		Diluent (pos DIL)	0µL
		Reagent (nacelle 2)	0µL
		Washing (at the end of loading)	0
2 <sup>nd</sup> loading step	YES	Incubation time (T1)	0 s
		Diluent (pos DIL)	0µL
		Specimen	0µL
		Washing between specimen	0
		Reagent (nacelle 3)(calcium thromboplastin)	75µL
		Washing after reagent loading	2
		Mix time (T2)	120 s
		Activation time (T3)	0 s
3 <sup>rd</sup> loading step	NO	Reagent (pos 17)	0µL
		Washing after reagent loading	0
		Mix time (T4)	0 s
Washing	NO	Activation time (T5)	0 s
		Reagent (pos 1)	0µL
		Reagent (pos 16)	0µL
Levels detection	NO	Cycles number	1
		Washing at the end of loading	1
Acquisition parameters	YES	Ramp	yes
		Inter-ramp interval (T6)	3s
		Delay before acquisition (T7)	0s
		Acquisition time (T8)	350s
		Canal used	660nm
		Rotation speed	1200 rpm

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