Manufactured By: HYPHEN BioMed

HEMOCLOT FVII reagent ON STA-R

Adaptation of HEMOCLOT FVII reagent on STA-R

1. Reconstitution of HEMOCLOT FVII Def (Ref ACK081K) reagent .

Determination of factor VII activity with a clotting method

NAME	Reconstitution	Stability	Stabilization at room Temperature
Hemoclot VII def Reagent	1 ml Distilled water(*)	72 hours 2-8°C 24 hours at room T° 1 month frozen at -20°C	30 mn before any use (**)

Reconstitution:

(*) After reconstitution with distilled water, let the factor reagent to stabilize for 15 minutes at room temperature.

Storage of reagents:

Take care of putting up the specific caps back on the bottles before storing them at 2°-8° C and of strictly respecting the temperature stabilization time of 30 minutes before using the reagents on the automate.

Stabilization of reagents:

(**) It is necessary to let the reagent temperature to stabilize for at least 30 minutes on the automate before any use.

2. Preparation of the calibrators and controls

Calibration curve:

- Prepare 1 ml of normal human pooled plasma (by definition the normal citrated plasma pool corresponds to a concentration of 100% of Factor VII when diluted 1:10 in Owren Koller buffer by the automate).

Or

- The calibration curve can also be established with the BIOPHEN Plasma Calibrator (# 222101), using the factor VII concentration indicated for the lot used.

Name	Reconstitution	Stability	Stabilization in T°
CALIBRATION Biophen Plasma Calibrator (ref A222101)	1 ml of distilled water(*)	24 hours at 2° - 8° C 8 hours at Room T°	30 minutes <i>on STAR board</i> before any use (**)

Controls

Name	Reconstitution	Stability	Stabilization in T°
<u>Biophen Normal Control</u> (ref A223201) <u>Biophen Abnormal Control</u> (ref A223301)	1 ml of distilled water(*)	24 hours at 2° - 8° C 8 hours at Room T°	30 minutes <i>on STAR board</i> before any use (**)

Reconstitution:

(*) After reconstitution with distilled water, let the reagent to stabilize for 30 minutes at room temperature.

Stabilization of reagents:

(**) It is necessary to let the reagent temperature to stabilize for at least 30 minutes on the automate before any use.

Storage of reagents:

Take care of putting up the specific caps back on the bottles before storing them at 2°-8° C and of strictly respecting the temperature stabilization time of 30 minutes before using the reagents on the automate.

Do not freeze calibrators and controls

Nota : A calibration curve must be carried out of each new batch of Hemoclot FVII reagent; after each important maintenance of the instrument, or when measured values for controls are out of the acceptance range.

NOTA: Plasmas used for the reagent and calibrator/controls were tested with registered methods and found negative for HIV antibodies, HBs Ag and HVC antibodies. However, no assay may warrant the total absence of infectious agents. Any product of human origin must then be handled with all the required cautions as being potentially infectious.

Results:

The calibration curve is of the Log (Clot time) -Log (% activity VII) type.

The values obtained for the patients and controls are directly calculated from the calibration curve (when tested at the standard dilution).

The results are expressed in %FVII activity.

5. Programming of the STA R analyser

Click on the icon **set up software** for the **manager program** and create the program according to:

TESTS				Configuration			
Method							
IDENTIFICATION							
Abréviation			HEM VII		Last up date.		
Name		HEMOCLOT FVII			Method		CLOT-BASED
SAMPLE			DILUENT				
Vol. µl	Incu.	Dilution	Id.	Name	Stab. h	Min. Vol.	
50 µl	0 sec	1/10	11361	OWREN KOLLER	72	0.90	
REAGENTS							
	Id.	Name	Stab. h	Vol. µl	Incu. sec	Prec.	Continued
Ra	R1	R1 VII DEF	8	50	240	<input type="checkbox"/>	
Rb						<input type="checkbox"/>	
Rc						<input type="checkbox"/>	
THR	THROMBO	THR	24	75			

Enter data concerning washing

	Id.	Nom	Stab. h	Vol. µl	Incu. sec	Prec.	Vial. ml	Min Vol. ml
Diluant	11361	OWREN-KOLLER	72				15	0.9
Ra	R1	R1 VII DEF	8	50	240	<input type="checkbox"/>	1	0.2
Rb						<input type="checkbox"/>		
Rc						<input type="checkbox"/>		
Rd	THR	THROMBO	24	75			6	0.5
Washing								
	Ra		Rb		Rc		Rd	
Before	No						No	
After	Normal						Intensive	
Wash								
Name								
Stab. H								
Vial ml								

Click on the icon **Result software** for the **manager program** and create the program according to:

TESTS		Configuration	
I. Result			
METHOD: CLOT-BASED		RESULT	
Min. Time	10	Primary Units	%
Max Time	300	Corrector	1.00
Mean Time	40		
Clot		Determination	
<input type="radio"/> Normal		<input type="radio"/> Single	
<input type="radio"/> weak		<input type="radio"/> Duplicate	
<input type="radio"/> Mean			
<input checked="" type="checkbox"/> Rd Heating		Precision (%) 5.00	
<input checked="" type="checkbox"/> Stirring		Redilution Condition	
		1/ <	
		1/ >	
		VALIDATION (%)	
		Min.	Max.
		5	120

Write calibration data.

TESTS		Configuration			
II. Calibration					
MODE : Linear		Détermination			
		<input checked="" type="checkbox"/> Duplicate <input type="checkbox"/> Single			
SCALE		Concentrations			
RAW DATA : <input type="checkbox"/> Linear <input checked="" type="checkbox"/> Log		<input type="checkbox"/> Linear <input type="checkbox"/> Inverse <input checked="" type="checkbox"/> Log			
	Id.	Key	Name	Stab. H	Dilution
Calibrator 1	BPC		BPCVII	*	1/10
Calibrator 2	BPC		BPCVII	*	1/20
Calibrator 3	BPC		BPCVII	*	1/40
Calibrator 4	BPC		BPCVII	*	1/80
Calibrator 5	BPC		BPCVII	*	1/160
Ctrl, Niv. 1					
Ctrl, Niv. 2					

*according to the calibrator used

Click on the icon **Printout/Transmission software** for the **manager program** and create the program according to:

TESTS		Configuration				
III.		Printout/Trans.				
PARAMETERS						
	Unit	Factors. Conver	Print	Transmission Test number	Usual Values Min. Max.	
Main	%		<input checked="" type="checkbox"/>	0	60	140
Aux1	SEC		<input checked="" type="checkbox"/>	0		
Aux2			<input type="checkbox"/>			
Aux3			<input type="checkbox"/>			
Printout Limits						
Min.	<input type="text" value="1"/>	Max.	<input type="text" value="150"/>			

Show the last page and enter data concerning the quality control.

TESTS		Configuration				
		Q.C				
LEVEL1						
	Id.	Key	Name	Stab. h	Vial. ml	Min Vol ml
Control	BNC		BN CONT	*	*	*
LEVEL 2						
	Id.	Key	Name	Stab. h	Vial. ml	Min Vol ml
Control	BAC		BA CONT	*	*	*
LEVEL3						
	Id.	Key	Name	Stab. h	Vial. ml	Min Vol ml
Control						
Period : <input type="text"/> hours <input type="text"/> <input type="checkbox"/> Tests <input type="checkbox"/> Vial						

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