



Manufactured By: HYPHEN BioMed

Calcium Chloride 0,025M (M/40)
Chlorure de Calcium 0,025M (M/40)
(Set of 3x25 ml or 12x25ml/ Coffret de 3x25 ml ou 12x25 ml)

Ref. AAR001A / AAR001K

STORE AT
2-8°C

For in vitro diagnostic use only



Diagnostic in vitro exclusivement

CONSERVER A
2-8°C

Composition:

Solution containing Calcium Chloride (0.025M) and Sodium azide (0.9 g/l) as preservative.

***CAUTION:** Sodium azide (NaN₃), may react with lead and copper plumbing to form highly explosive metal azides. In order to avoid this risk, flush with large volumes of water when discarding into a sink.

Ready to use.

Presentation:

25 ml vials, ready to use.

AAR001A: 3 x 25 ml

AAR001K: 12 x 25 ml

Use:

Calcium Chloride solution for coagulation assays, such as the Activated Partial Thromboplastin Time (APTT) or for the assays of factors of the intrinsic coagulation pathway.

Ready to use.

Note 1: Take care of stabilizing the solution at room temperature (18-25°C) for 30 minutes before use.

Note 2: Refer to the specific associated kit inserts for instructions concerning the tests to be performed.

For in vitro use only.

Stability:

In its original vial, unopened and kept at 2-8°C, until the expiration date printed on the label.

When open and protected from any contamination, this solution is stable for 4 weeks at 2-8°C.

Composition :

Solution contenant du chlorure de calcium (0,025M) et de l'azoture de sodium (0,9 g/l) comme conservateur.

***PRÉCAUTIONS :** L'azide de sodium (NaN₃) peut générer des composants explosifs au contact des canalisations en plomb ou en cuivre. Pour éviter ce risque, effectuer des lavages intensifs.

Prêt à l'emploi.

Présentation :

Flacons de 25 ml, prêts à l'emploi.

AAR001A : 3 x 25 ml

AAR001K : 12 x 25 ml

Utilisation:

Solution calcique pour dosages de coagulation, tels que temps de céphaline + activateur (TCA) ou dosages de facteurs de la voie endogène .

Prêt à l'emploi.

Note 1: Veiller à bien stabiliser la solution à température ambiante (18-25°C) pendant 30 minutes avant utilisation.

Note 2 : Se référer aux notices spécifiques des kits utilisés pour les instructions concernant le test à effectuer.

Utilisation *in vitro* exclusivement.

Stabilité du flacon:

Dans son flacon d'origine, fermé et conservé à 2-8°C, jusqu'à la date de péremption imprimée sur l'étiquette.

Après ouverture et protégée de toute contamination, cette solution est stable 4 semaines à 2-8°C.

Rev 080211
D.750/08/AR/001



6560 Gove Court - Mason, OH 45040

Phone: 513.770.1991

Toll Free: 866.783.3797

Fax: 513.573.9241

Email: info@aniara.com

www.aniara.com