# PLASMA PAI-1 CONTROL

## # ASC011K/E

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Last revision / Dernière version: 17/08/2006

### Intended use

Lyophilised plasmas, at defined PAI-1: Ag concentrations, as measured with the ELITEST-PAI-1 : Ag kit (ref ACK102A), for the quality control of PAI-1 antigenic assays.

### Reagents:

Each kit contains:

- 3 vials of 1 ml of normal plasma, lyophilised, at a high PAI-1-Ag concentration (Control I).
- 3 vials of 1 ml of normal plasma, lyophilised, at a low PAI-1-Ag concentration (Control II).

The PAI-1: Ag concentrations and the acceptance ranges for controls I and II, can present variations from lot to lot, but are indicated on the flyer provided with each set of controls.

### Reagent Preparation and Stability:

In their original package, and before any use, when stored at 2-8°C, the reagents are stable until the expiration date printed on the kit.

### Excipients:

Glycine, stabilizers.

### Preparation:

- **PAI-1 Control I (high):** restore the vial with 1 ml distilled water; let for 15 min at room temperature; mix gently until complete dissolution of the content.
- **PAI-1 Control II (low):** restore the vial with 1 ml distilled water; let for 15 min at room temperature; mix gently until complete dissolution of the content.

These plasmas are stable:

- 24 hours at room temperature (18-25 °C)
- 48 hours at 2-8 °C
- 2 months, frozen in their original vial, or in a plastic tube (before use, thaw in a water bath at 37 °C, for at least 15 min).

**Nota :** Plasmas used for PAI-1 controls were tested with registered methods and found negative for HIV antibodies, HBs Ag and HVC antibodies. However, no assay may warrant the total absence of infectious agents. Any product of human origin must then be handled with all the required cautions, as being potentially infectious.

### Application

The PAI-1: Ag concentrations, indicated on the flyer, for Control I and Control II, have been measured with the ELITEST PAI-1: Ag (ACK102A) kit.

These assays are standardised by reference to the NIBSC international standard.

The target values and the acceptance ranges, indicated on the flyer, are valid for ELITEST PAI-1: Ag. The results obtained can vary, when these controls are used with other kits according to the assay reactivity and its standardisation. These controls can nevertheless be used with different kits for testing run to run variability, but the acceptable values must be determined by each laboratory.

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