AT- Tris buffer –Anti Ila
(Progressive Antithrombin)
Tampon AT-Tris-Anti Ila
(Antithrombine Progressive)
(Set of 1x10 mL / Coffret de 1x10 mL)

Ref. AR104A

STORE AT
2-8°C

For *in vitro* research use only

Recherche *in vitro* exclusivement

CONSERVER A
2-8°C

Lot:

English, last revision 08-2017

Expiration:

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COMPOSITION:

Buffer containing Tris, sodium chloride and Sodium azide (as preservative), proposed for measuring the progressive Antithrombin activity with an Anti Ila method (and more especially variant protocol of Biophen AT (Anti Ila) #221122), in the absence of Heparin. The specific protocol is available upon request (D.750.30/AT-prog/antilla).

***CAUTION: Sodium azide (NaN₃), may react with lead and copper plumbing to form highly explosive metal azides. Flush with large volumes of water when discarding into a sink.**

PRESENTATION:

10 mL vials, ready to use.
AR104A: 1 x 10 mL.

PH:

8.40.

USE:

Dilution buffer for the measurement of Progressive Antithrombin activity (Anti Ila) in plasma, using an Anti Ila method, and a variant protocol of Biophen AT (Anti Ila) kit (#221122), performed in the absence of Heparin. Ready to use

Note 1: Stabilize the solution at room temperature (18-25°C) for 30 minutes before use.

Note 2: Refer to the specific associated kit inserts or protocols for instructions concerning the tests to be performed.

For *in vitro* use only.

STABILITY:

In its original vial, unopened and kept at 2-8°C, until the expiration date printed on the label.

When open and protected from any contamination, this solution is stable for 4 weeks at 2-8°C.

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