



ANIARA

Manufactured By: HYPHEN BioMed

BIOPHEN DiXaI : Applications and principle



Ref. ID# A221030

This Product is CE Marked.

In US and Canada: For Research Use Only. Not for Use in Diagnostic Procedures.

DiXaI quantitative chromogenic assay (plasma or purified milieu).

R1: Human FXa
R2: FXa substrate
R3: Buffer

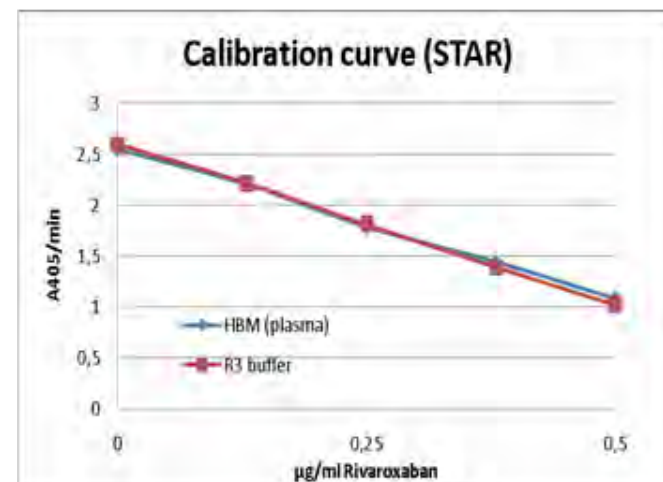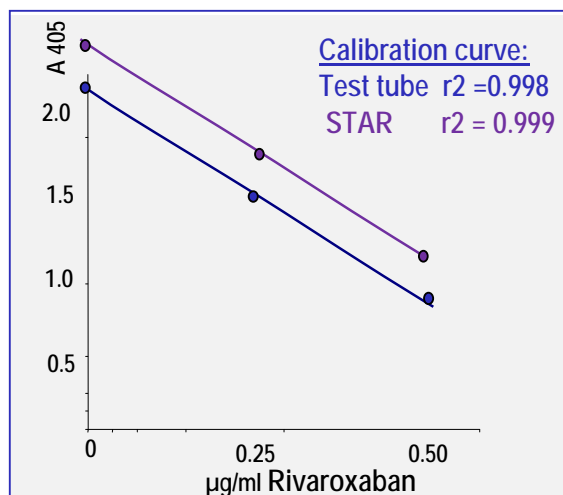

Form AH119
04-2012

ANIARA

7768 Service Center Drive • West Chester OH 45069

Phone: 513.770.1991

Toll Free: 866.783.3797


Fax: 513.573.9241

Email: info@aniara.com

www.aniara.com



BIOPHEN DiXaI proves excellent SPECIFICITY (microplate data):

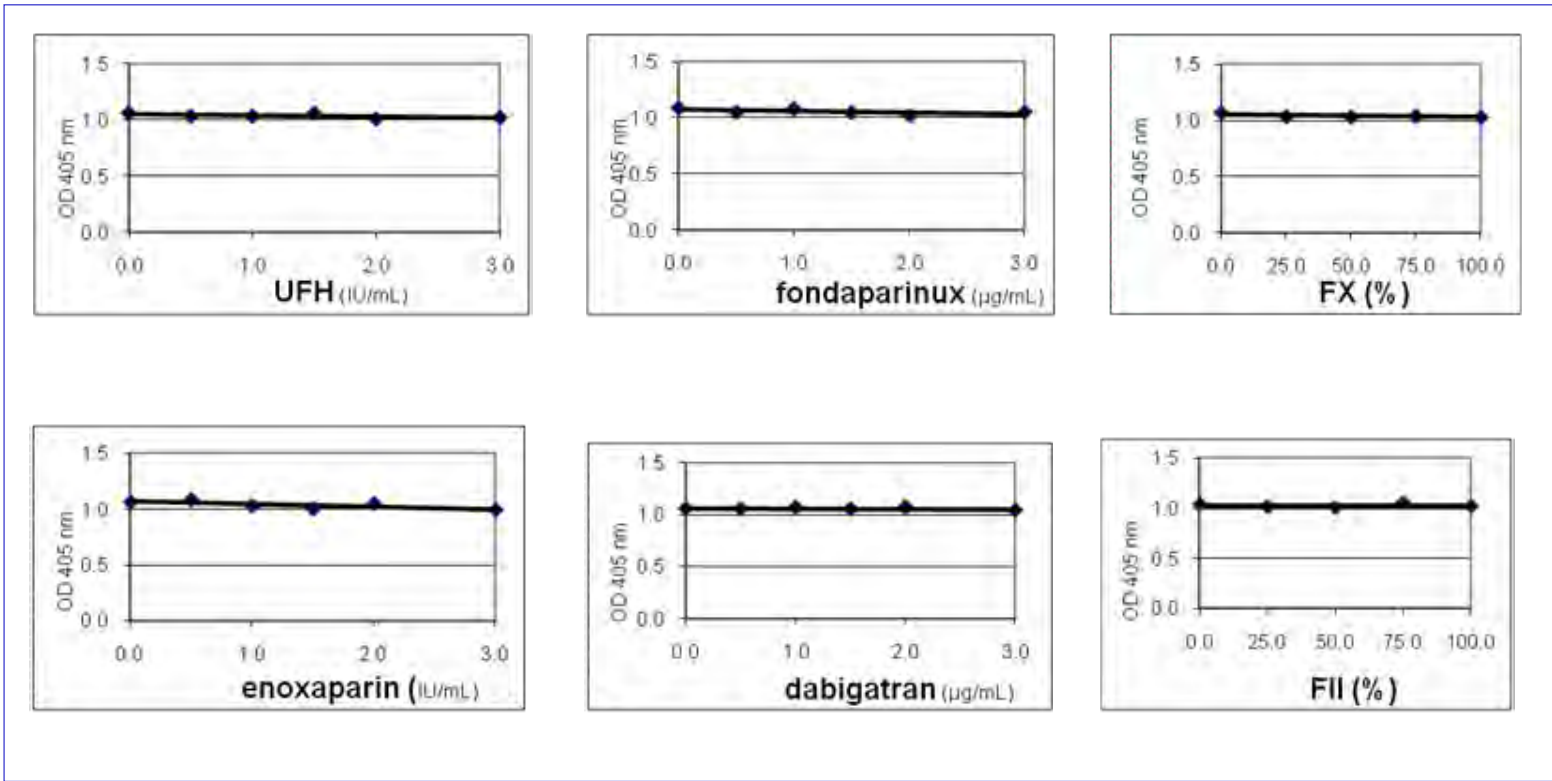

Form AH119
04-2012



7768 Service Center Drive • West Chester OH 45069

Phone: 513.770.1991

Toll Free: 866.783.3797

Fax: 513.573.9241

Email: info@aniara.com

www.aniara.com



Biophen DiXal: Reproducibility and recovery study:

STAR (N=10):	Intra assay CV	Inter assay CV
0.20 $\mu\text{g/ml}$	2.5%	14.4%
0.40 $\mu\text{g/ml}$	1.7%	10.8%

Rivaroxaban RECOVERY	+0 μM	+0.50 μM
Mean recovered μM (N=10)	0.01	0.47
SD	0.011	0.017
Min-Max	0-0.03	0.44 - 0.49

Conclusion:

Specific, flexible, robust.

⇒ High potential tool for new DiXals