

Anti-FXa activity of Apixaban

Product Ref. N°

BIOPHEN™ Apixaban Control Low / Control (A225201-RUO/A225301-RUO)

Kit presentation

BIOPHEN™ Apixaban Calibrator Low / Calibrator (A226101-RUO/A226201-RUO)

Status

Set of plasma calibrators (3 levels) and quality controls (2 levels) for Apixaban measurements by chromogenic method.

RUO

Intended use:

A set of lyophilized human plasmas supplemented with Apixaban at different concentrations for measuring its anti-FXa activity. One stage (BIOPHEN™ Heparin LRT) and two stages (BIOPHEN™ DiXal) Anti-FXa chromogenic methods are used with specific Apixaban calibrators and controls Low (0 to 120 ng/mL) and Standard (0 to 600 ng/mL), calibration ranges are developed for measuring Anti-FXa activity of Apixaban in plasma.

Assay principle:

With BIOPHEN™ DiXal:

This anti-FXa method is based on the inhibition of a constant and in excess quantity of Factor Xa (FXa) by Apixaban. The residual FXa hydrolyses the FXa-specific chromogenic substrate, releasing paranitroaniline (pNa) measured by absorbance at 405 nm.

With BIOPHEN™ Heparin (LRT):

This method is a kinetics assay based on the inhibition of a constant and in excess quantity of Factor Xa (FXa) by Apixaban, and simultaneously the excess of FXa hydrolyses FXa-specific chromogenic substrate, releasing paranitroaniline (pNa) measured at 405 nm.



Key Characteristics & Advantages:

Apixaban in practice

Research Applications:

- Apixaban is an oral anticoagulant drug, used for curative or preventive indications. Although monitoring of Apixaban levels is not necessary, when required, measuring the Apixaban concentration can be helpful in some research applications.
- BIOPHEN™ Apixaban Calibrator (Standard/Low) is proposed to plasma using anti-Xa chromogenic assays.
- BIOPHEN™ Apixaban Control plasmas are proposed for the quality control of Apixaban measurements in plasma using anti-Xa chromogenic assays.

Market requirements & challenges:

- Standardised automated assay, with good stability, safe and effective, polyvalent for use on major automated coagulation analyzers.
- BIOPHEN™ Apixaban Calibrator and Control plasmas are qualified against an Internal Reference Standard, whose qualification is linked to the reference method by LC-MS/MS.

Dedicated methods to support your needs

Easy and fast Automation:

- Simple, rapid (3 to 5min), fully automatable on any coagulation analyzer,
- Method applications in combination with anti-FXa reagents: Sysmex CS-series, STA-R® series, ACL TOP® family and BCS® XP.

Optimized consumption with high stability for Apixaban calibrators and controls

- 24 hours onboard (CS series, STA-R® series, ..) ; 7 days at 2-8°C; 6 months at -20°C
- Long shelf life at 2-8°C in original packaging

Extended measurement with two ranges, for the various clinical situations

Low range: from about 10 to 120 ng/mL of Apixaban
Standard range: from about 20 to 600 ng/mL of Apixaban


Excellent Precision with low CVs:

- Within-run ≤ 5% (CS-5100)
- Between-run ≤ 7% (CS-5100)

Specific and reliable assay, without interferences related to sample content
(Hemoglobin, bilirubin, triglycerides, heparins, Dabigatran)

Safe and standardized method for optimal testing

- Good lot-to-lot consistency; High quality raw materials used
- Traceability calibrators/controls, qualified against an Internal Reference Standard and LC:MS/MS

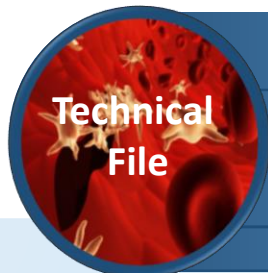

Anti-FXa activity of Apixaban

Product Ref. N°

BIOPHEN™ Apixaban Control Low / Control (A225201-RUO/A225301-RUO)

Kit presentation

BIOPHEN™ Apixaban Calibrator Low / Calibrator (A226101-RUO/A226201-RUO)
Set of plasma calibrators (3 levels) and quality controls (2 levels)
for Apixaban measurements by chromogenic method.





Status

RUO

Assay Characteristics*:

Multiplatform, easy to use, stable, low CVs , no interferences

Calibration curves:

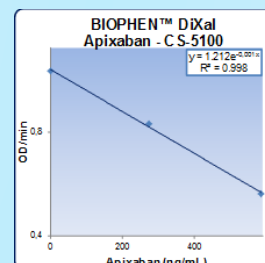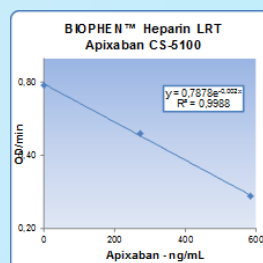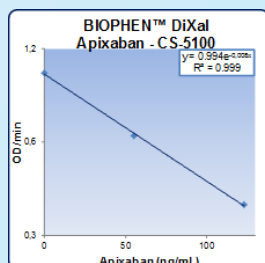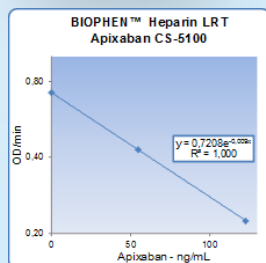

Precision:

Apixaban Low Range

Apixaban Standard Range

Low Range		BIOPHEN™ Heparin LRT				BIOPHEN™ DiXal			
		CS-5100	STA-R®	ACL TOP®	BCS® XP	CS-5100	STA-R®	ACL TOP®	BCS® XP
Precision intra-assay	n	30	30	30	10	30	30	30	10
	CV% (QC1L)	2.5	3.2	3.2	5.2	3.8	5.5	6.0	2.8
	CV% (QC2L)	1.2	2.4	3.6	2.0	1.8	1.8	2.9	1.5
Precision inter-assays	n	20	12	30	10	20	12	30	10
	CV% (QC1L)	3.8	6.8	4.7	7.7	4.5	9.1	6.5	10.5
	CV% (QC2L)	2.2	4.0	4.4	2.8	2.9	5.9	5.5	5.1



Standard Range		BIOPHEN™ Heparin LRT				BIOPHEN™ DiXal			
		CS-5100	STA-R®	ACL TOP®	BCS® XP	CS-5100	STA-R®	ACL TOP®	BCS® XP
Precision intra-assay	n	30	30	30	10	30	30	30	10
	CV% (QC1L)	1.2	2.4	5.0	2.9	3.4	3.7	4.5	3.5
	CV% (QC2L)	0.9	1.8	4.4	1.3	1.8	1.5	4.2	3.1
Precision inter-assays	n	20	20	30	12	20	20	30	11
	CV% (QC1L)	2.6	5.2	5.6	5.1	3.5	4.9	3.0	11.8
	CV% (QC2L)	1.9	3.5	4.1	2.8	3.2	4.1	4.2	6.5

Methods & analyzers Comparison:

Example: LC:MS vs Anti-Xa assay

Apixaban (ng/mL)	Chromogenic (HBM anti-Xa assay)	LC:MS
Level 1	<LLOQ	<LLOQ
Level 2	46	45
Level 3	93	88
Level 4	263	268
Level 5	537	526
Level 6	635	617

Anti-Xa assays

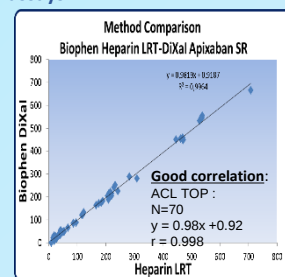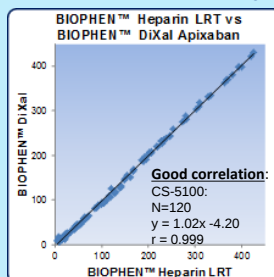

* The values cited for the performance characteristics of the assay represent example of typical results and are not to be regarded as specifications for the test

Stability for Calibrators and Controls:

Shelf life in original packaging

30 months at 2-8°C

Onboard

24 hours on CS series, STA-R®...

After Opening

7 days at 2-8°C

48 hours at room temperature (18-25°C)

6 months at -20°C or less

Interferences:

No interference of usual interferences up to

10 to 80 mg/dL Bilirubin, 150 to 1000 mg/dL Hb, 600 to 1200 mg/dL Intralipids and 400 ng/mL Dabigatran.

Related products:

Needed	Product name	Ref N°:
Reagents	BIOPHEN™ DiXal BIOPHEN™ Heparin LRT BIOPHEN™ Heparin	A221030-RUO A221011-RUO/A221013-RUO/A221015-RUO A221003/A221006

Kit presentation:

A225301-RUO: 2 levels, 6 x 1 mL each
A226201-RUO: 3 levels, 4 x 1 mL eachA225201-RUO: 2 levels, 6 x 1 mL each
A226101-RUO: 3 levels, 4 x 1 mL each