



ANIARA

DIAGNOSTICA, INC.

ReaScan[®] + Revolution

Welcome to the ReaScan Revolution

Proven lateral-flow technology and proprietary assay development techniques are integrated into the next generation ReaScan+ product family. Its ease of use combined with result traceability and LIS connectivity add confidence to the results.



Reagent

For Research Use Only. Not For Use in Diagnostic Procedures.

ReaScan[®]+ Research Products

The ReaScan Revolution

ReaScan+ reader is a compact rapid test device with an intuitive graphical user interface. It combines rapid and reliable results with ease of use. ReaScan+ reader offers the option of Immediate Reading or Incubation Mode (walk away) allowing flexible workflow in the laboratory.

Up to 300 results are stored into the internal non-volatile memory

Touch screen with an easy to use User Interface supporting multiple languages

The 2D code on ReaScan cassettes is automatically scanned to verify the correctness of the test

User and Sample ID, date and time and test kit batch number are stored with each result for maximum traceability



Results can be automatically transferred to the LIS or laboratory middleware

Optional 1D/2D barcode scanner to automate the input of sample ID

ReaScan+ C6 LYME IgG

Reagent is a pioneer in rapid testing for zoonotic diseases and lately focused on tests for Lyme Borreliosis and Lyme neuroborreliosis. ReaScan+ C6 LYME IgG is a lateral flow rapid test intended for detection of *Borrelia burgdorferi* specific IgG antibodies in human serum and cerebrospinal fluid (CSF). The test is intended for use as an aid in Lyme Disease research.

ReaScan+ LYME IgM/IgG Combo

ReaScan+LYME IgM/IgGCombo is a lateral flow rapid test intended for qualitative detection of *Borrelia burgdorferi* specific antibodies in human serum and cerebrospinal fluid (CSF). The test uses C6 (VlsE) and PepC10 (OspC) antigens in separated test lines. PepC10 generates an early IgM response and thus it is likely to react especially in acute Lyme disease.

Product	Test performance	Packaging	Ref. number
ReaScan+ C6 LYME IgG	Specificity 99 % (CSF), 95 % (serum) Sensitivity 97 % (CSF), 97 % (serum)	10 tests	114313
ReaScan+ LYME IgM/IgG Combo	PPA 94,8 % (CSF, serum/plasma) NPA 95,6 % (CSF, serum/plasma)	10 tests	115103

The sensitivity of the ReaScan+ C6 LYME IgG was determined by analyzing 108 paired serum and CSF samples with definite LNB. The specificity of the ReaScan+ C6 LYME IgG was evaluated by analyzing paired serum and CSF samples from 104 non-LNB. EQV results counted as positive. ReaScan+ LYME IgM/IgG Combo PPA and NPA values for 326 samples including 168 positive serum/plasma samples, 117 negative serum/plasma

samples, 23 positive CSF samples and 18 negative CSF samples. EQV ReaScan+ LYME IgM/IgG Combo results were calculated as negatives.

ReaScan CXCL13

ReaScan CXCL13 is a lateral flow rapid test for semiquantitative determination of CXCL13 in human cerebrospinal fluid CSF, detecting Lyme neuroborreliosis (LNB) in research settings.

Product	Test performance	Packaging	Ref. number
ReaScan CXCL13	Specificity 96 % Sensitivity 100 % with cut-off 250 pg/mL	10 tests	114253

Performance was obtained by comparing ReaScan CXCL13 with a widely used CXCL13 ELISA (Human CXCL13/BLC/BCA-1 Quantikine, R&D Systems, Minneapolis, USA) with 225 samples with suspected LNB. Ref.: Pietikäinen A, et al, Point-of-care testing for CXCL13 in Lyme neuroborreliosis, *Diagn Microbiol Infect Dis* (2018), <https://doi.org/10.1016/j.diagmicrobio.2018.02.013>.

ReaScan TBE IgM

ReaScan TBE IgM is a lateral flow rapid test for qualitative detection of tick-borne encephalitis (TBE) virus specific IgM antibodies in human serum and cerebrospinal fluid (CSF) in research settings.

Product	Test performance	Packaging	Ref. number
ReaScan TBE IgM	Specificity 100 % (CSF), 98 % (serum) Sensitivity 92 % (CSF), 99 % (serum)	10 tests	114106

Performance was evaluated with 172 serum samples of confirmed TBE and 141 TBEV IgM negative serum samples. See Albinsson B et al. 2020. Multi-laboratory evaluation of ReaScan TBE IgM rapid test, 2016 to 2017. *Euro Surveill.* 2020 Mar;25(12).

ReaScan Hantavirus

Reagent is an undisputed leader in Hantavirus rapid testing. Reagent offers rapid tests intended for qualitative detection for Puumala virus and Dobrava or Hantaan virus specific IgM antibodies from human serum or plasma.

Product	Test performance	Packaging	Ref. number
ReaScan+ PUUMALA IgM	Specificity 99 % Sensitivity 98 %	10 tests	115105
ReaScan DOBRAVA-HANTAAN IgM	Specificity 97 % Sensitivity 95 %	10 tests	114102

Performance of each test was obtained when evaluating test performance with panels of confirmed samples.

For Research Use Only. Not For Use in Diagnostic Procedures.

ReaScan[®]+

Technical data

Display: 4.3" Touch screen
Memory: 300 data sets
Connectivity: 3x USB ports, 1x LAN port
Power Supply: 12 Vdc (adapter included)
Size: 140 x 180 x 150 mm
Weight: 1150 g
Operating temperature: +15°C ... +35°C
One year limited warranty



Feature highlights

- Fast, accurate and objective results
- Traceability of test results
- Robust and maintenance free reader
- Open and flexible connection to different LIS systems Intuitive
- Graphical User Interface Immediate reading mode or delayed reading after incubation time

Order information

Product name	Ref. number
ReaScan [®] reader	105100
ReaScan [®] connect server	105130
Zebra DS2208 SR USB-Kit Black barcode scanner	105110
Dymo Label Writer 450 printer	105120

For Research Use Only. Not For Use in Diagnostic Procedures.

ANIARA
DIAGNOSTICA, INC.

7768 Service
Center Dr.,
West Chester,
OH 45069

+1 513 770 1991
info@aniara.com
www.aniara.com

R-002-00
2022-06-02